**QuikRead® FOB Positive Control, Cat. No. 06027**

## Intended use

QuikRead® FOB Positive Control is intended for quality control of the QuikRead® FOB quantitative and the QuikRead go® iFOBT assays.
**For *in vitro* diagnostic use.**

## Principles of the procedure

Use two drops of QuikRead FOB Positive Control solution instead of an unknown, prediluted faecal sample. Continue by following the instructions for the assay procedure.

## Reagents

QuikRead FOB Positive Control contains human haemoglobin. It should be handled in the same way as patient samples.

## Preparation

Allow QuikRead FOB Positive Control to reach room temperature (18...25°C) before use. Mix well without shaking. QuikRead FOB Positive Control is now ready for use.

## Storage

QuikRead FOB Positive Control is stable until the expiry date when stored at 2...8°C. Discard any vial with signs of microbial growth.

## Expected values

With QuikRead go software version 6.1.5 or later, the result should be >1000 ng/ml and/or >200 µg/g for the quantitative test. For the qualitative test the result should be "POSITIVE".

With QuikRead 101 software version 9.0 or later using QuikRead FOB quantitative kit (Cat. No.138820), the result should be "POSITIVE FOB >1000 ng/ml". With earlier software versions the result should be "POSITIVE FOB". In case of a result <75 ng/ml and/or <15 µg/g for the quantita-

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**QuikRead® FOB Positive Control, Ref. No. 06027**

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tive QuikRead go iFOBT test, or "NEGATIVE"/"NEGATIVE FOB" for the qualitative QuikRead go iFOBT/QuikRead FOB Quantitative test, please verify that the kit reagents and the magnetic card are from the same lot and that none of the reagents have exceeded their shelf-life. Repeat the test. If the result is still negative, please contact your local QuikRead distributor.

## Warnings and precautions

- The QuikRead FOB Positive Control contains material of human origin, which has been tested and found negative for human immunodeficiency virus antibodies (HIV 1 and 2), hepatitis C virus (HCV) and hepatitis B surface antigen (HBsAg). As a precaution, handle it as if capable of transmitting infectious agents.
- Reagents contain <0.1% sodium azide, which is not considered a harmful concentration.
- Sodium azide liberates toxic gas when in contact with acids. Azides may react with metal plumbing, forming explosive compounds. Azide build-up can be avoided by flushing with a large volume of water when disposing of the reagents.
- Do not use the product after the expiry date marked on the outer package.

## Disposal

- Dispose of contents according to national and local law.
- Materials of the components
Paper: Instructions for use
Cardboard: Outer box
Plastic: Control vials, vial caps
- When used in accordance with Good Laboratory Practice, good occupational hygiene and the instructions for use, the reagents supplied should not present a hazard to health.

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