

QuikRead[®] U-ALB

Brief description of the test procedure




1

Preparation:
Read the card.



2

Dispense 1 ml of buffer into
the cuvette.

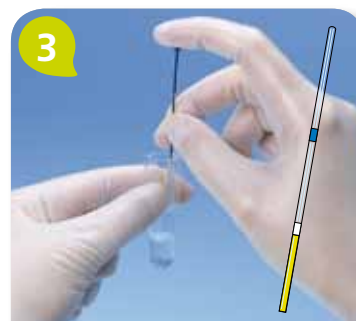

3

Add 20 µl of urine.



4

Put the cap on and mix gently
(not upside-down). Put the
cuvette into the measurement
well.



5

Blank measurement max.
30 seconds. Add the U-ALB-
reagent by pressing down the
inner part of the cap.



6

Take out the cuvette and mix
vigorously (back and forth) to
dissolve the reagent.



7

Put the cuvette back into the
measurement well. The U-ALB
result is displayed within 2
minutes.

For
more detailed
information,
please consult the
instructions
for use.

POC